



STUDEX EAR PIERCING CERTIFICATION PROGRAM VIRGINIA BOARD FOR BARBERS & COSMETOLOGY

CERTIFICATION EXAM - FORM 2

Multiple Choice (make only one selection for each of the questions)

1. The only interchangeable part of the Universal Piercing Instrument is:

- a. The plunger
- b. The plunger knob
- c. The interchangeable adapter

2. The customer's ear can be marked before piercing with:

- a. Ball point pen
- b. Studex non – toxic marking pen
- c. An eyeliner make-up pen

3. You should never pierce through:

- a. A torn ear
- b. A cyst
- c. All of the above

4. An ear piercing instrument should never be used to pierce:

- a. Eyebrow
- b. Nose
- c. Navel (belly button)
- d. All of the above

5. The release form should be used if:

- a. A minor is accompanied by a parent or legal guardian
- b. The customer is under the age of 18
- c. All customers must sign a release form

6. Studex brand Piercing Studs:

- a. Are made of medical grade stainless steel
- b. Are 24k pure gold overlay
- c. Are non – allergenic
- d. All of the above

7. It is recommended that the piercing stud be left in the ear:

- a. For one week
- b. For 4 to 6 weeks
- c. Until the customer runs out of the ear care gel

8. With Studex brand piercing instruments you are able to pierce in:

- a. Surgical Stainless Steel
- b. 9k Gold
- c. 14k Gold
- d. 18k Gold
- e. Titanium
- f. All of the above

True / False

1. The pierced cartilage area of the ear will take longer to heal than a pierced earlobe.

- True
- False

2. You are only obligated to discuss "after ear care" with your customer if he or she asks.

- True
- False

3. Aftercare must be discussed, and customer provided "after care instructions" in writing, before piercing.

- True
- False

IMPORTANT: PLEASE READ AND SIGN BELOW

By signing this I acknowledge that I have read the operators manual for the Universal Piercing Instrument &/or System75 Ear Piercing Instrument and viewed the instructional video in it's entirety. I understand the importance of post ear piercing care and my responsibility to maintain clean and properly operating piercing equipment at all times.

Name _____ Birth Date ____/____/____

Certificate Issued to: _____
(Please print clearly)

Signature

Date ____/____/____

Please email to sales@studex.com
Fax to: 310-851-9400 or mail to:

Studex Corporation / Virginia Training
521 West Rosecrans Avenue, Gardena, CA 90248

Questions? Call 1-800-478-8339

Please follow directions on attached application for processing of your application.